

ProCare

Dent 11 A




Detergent

Powder, alkaline. For the automated reprocessing of dental instruments in Miele Professional thermal disinfectors.

ProCare Dent 11 A supports the removal of organic soil such as blood and saliva and prevents redeposition. Coarse residues from composites, dental cement and dental impression compound should be removed before they have had time to set.

ProCare Dent 11 A is not suitable for cleaning transmission instruments. The use of the ProCare Dent 10 MA liquid detergent is recommended for this purpose.


Quality features

ProCare Dent 11 A conforms to the EC directive 93/42/EEC, Annex I on medical devices. 

Product properties

- Excellent material compatibility
- Protection against corrosion for instruments and trays made from stainless steel

ProCare Dent 11 A is intended for commercial use only.

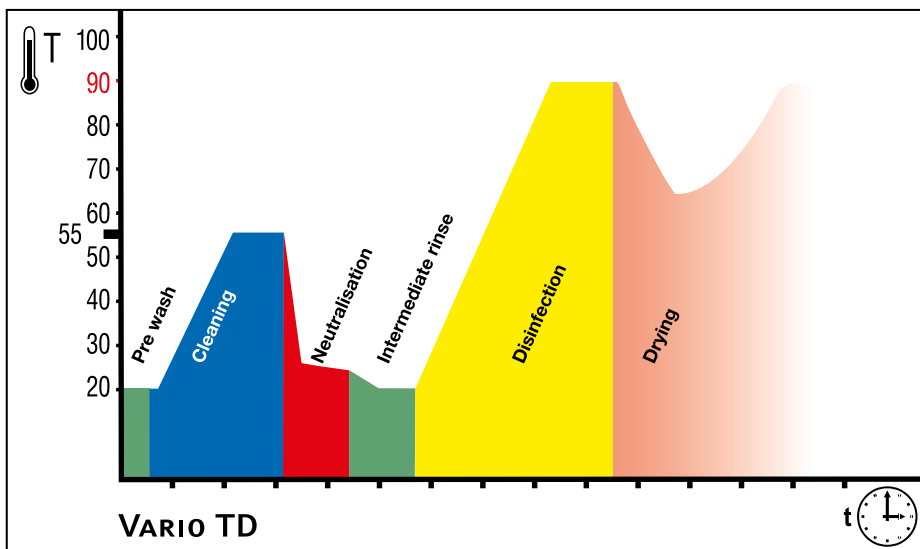

Use and dispensing

Miele ProCare Dent 11 A is formulated for use in processes in Miele Professional thermal disinfectors.

- Application concentration: 3 g/l via door dispenser for powder detergents
- varioTD programme

An identical quantity of detergent should be used each time, preferably by using the enclosed dispensing beaker.

The wash liquor should be completely rinsed off using water. No visible detergent residue should be present at the end of the process. On Miele Professional thermal disinfectors, this is guaranteed by the varioTD programme when properly used (e.g. correct loading).

Verification is provided by measuring the

increase of the conductivity in the final rinse during commissioning and routine maintenance. Miele recommends the use of fully demineralised water in the final rinse in order to protect the load and the washer-disinfector against stains and deposits and to protect anodised aluminium.

varioTD process

- Pre-wash with cold water to remove coarse soil (e.g. blood).
- Main wash with ProCare Dent 11 A at 55°C (dispensing temperature 40°C)
- Neutralisation
- Intermediate rinse with cold water
- Thermal disinfection at 93°C with 10-minute holding time and final rinse
- Drying

Please note:

- Do not mix with other products.
- Reprocessing should comply with all ordinances pursuant to the Medical Device Directive and should be performed with appropriate validated processes.
- Observe recommendations on reprocessing issued by instrument manufacturers with respect to the provisions of DIN EN ISO 17664.
- Operating instructions on Miele Professional thermal disinfectors must be observed.

ProCare

Dent 11 A




Technical data

- Bulk weight: 1015 to 1065 g/l
- pH-value (determined using fully demineralised water at 20°C) 3 g/l: approx. 11.1

Ingredients according to EC Regulation 648/2004 on detergents

- < 5% non-ionic tensides
- > 30% phosphates

Storage information

- Storage temperature: 0°C to 30°C / 32°F to 86°F
- Refit lid firmly after use. The product tends to get lumpy when exposed to dampness which can cause a loss of effectiveness.
- Can be used for 2 years under proper storage conditions
- Use-by date: See label next to hour-glass symbol 

Warning and Safety notes

- For safety and environmental information, cf. EC Safety Data Sheets. These are available from www.miele-professional.com.

Dispose of containers only when empty and sealed. Disposal of product residue: cf. Safety Data Sheet.

All information contained herein is current and correct to our knowledge. This does not however release users from obligations to perform their own trials and experiments. This cannot be deemed as a legal warranty of product characteristics or properties.

Available packaging unit

Designation	Packaging unit	Mat. no.
ProCare Dent 11 A	10 kg (22.0 lbs) bucket	9 051 940

Contact details:

Miele USA
9 Independence Way
Princeton, NJ 08540
800-991-9380
www.miele-professional.com



Chemische Fabrik
Dr. Weigert GmbH & Co. KG
Mühlenhagen 85
20539 Hamburg
www.drweigert.de